
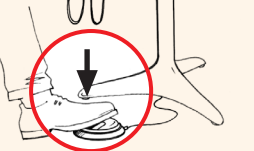
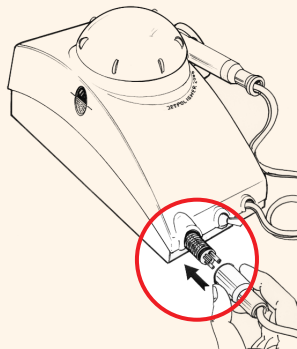
**INSTALLATION**


Check you have air and water from your unit 4 hole tubing

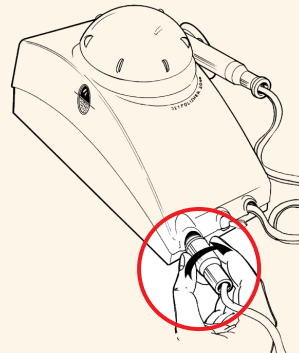Connect your 4-hole tubing to Polisher connector




screw up tight to prevent leakage

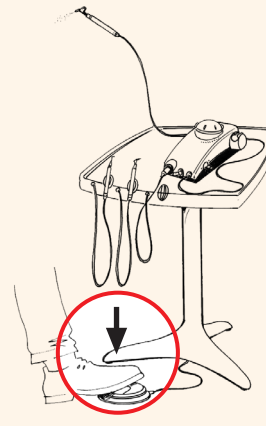
Your foot control will now activate the Polisher

**ADDING POWDER**

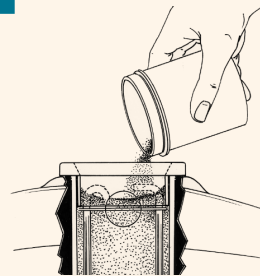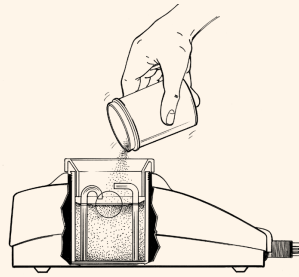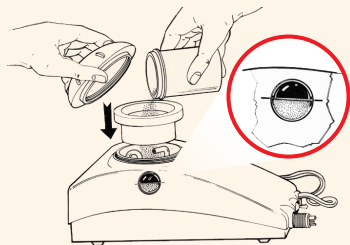Remember ! Do not add above fill line

Over filling will cause clogging

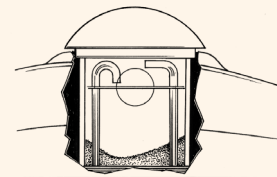
Less powder = less intense cleaning

**ADJUSTING THE CONTROLS**

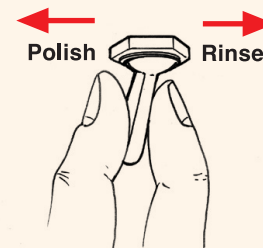Use the fingertip test to achieve correct powder/water ratio

Start at the lowest end and gradually build water intensity. Inadequate water will produce a spluttering, intermittent stream. Use only enough water for a steady stream.

Rinse/Polish mode selector  
Polish = water + air + powder  
Rinse = water + air only

**CORRECT ADJUSTMENT 'FINGERTIP' TEST**

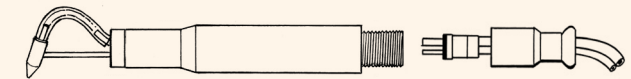
1. In the 'polishing mode', open the water control knob (H<sub>2</sub>O) fully.
2. Hold nozzle approximately 1 cm (3/8") from the finger tip.
3. Activate unit, and gradually reduce the amount of water in the stream by means of the water control knob.
4. A point will be reached when a spot of powder begins to collect on the fingertip. At this point inadequate water is present in the stream.
5. Increase the water flow until the spot of powder on the fingertip just disappears.

**TROUBLE SHOOTING**

**Unblocking a handpiece**


1. Disconnect handpiece

2. Depress foot pedal and check flow from handpiece connector

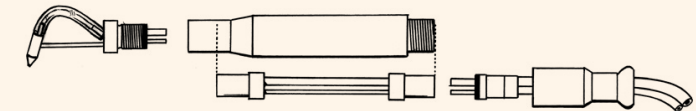3. Clean components in ultrasonic bath in warm water or mild vinegar solution


4. Dry thoroughly

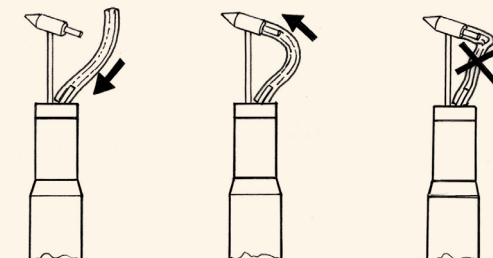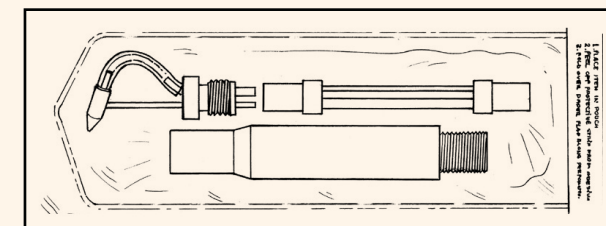
**Warning:**  
Always use a dry air supply. Drain water filter on your compressor daily.

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**DEL DENT** Ltd.